


MERIDIAN M110 INSTRUCTIONS FOR USE


SKIN PREPARATION FOR PATCH PLACEMENT




You will need NuPrep and gauze.

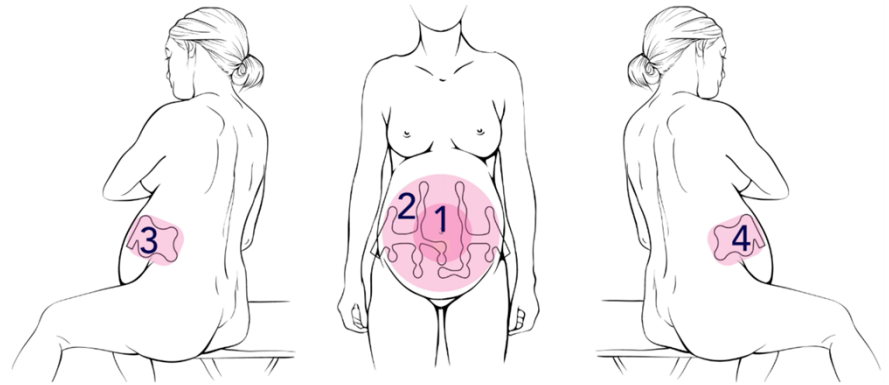



The four areas to be prepared for patch placement.

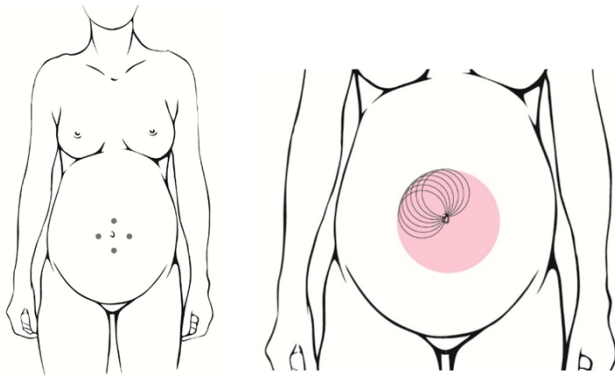

Place 4 dots of gel and rub with gauze into the pink area of skin with technique shown above.

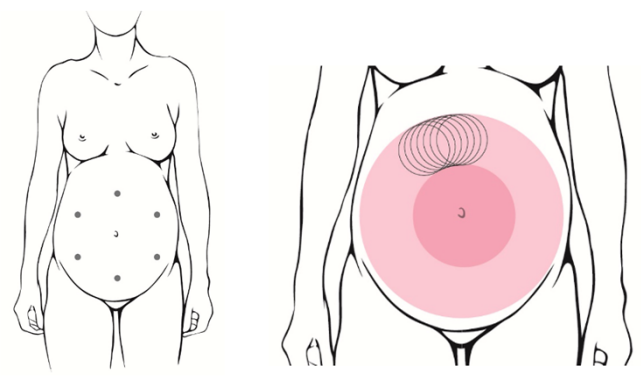

Place 6 dots of gel and rub with gauze into the pink area of skin with technique as shown above.

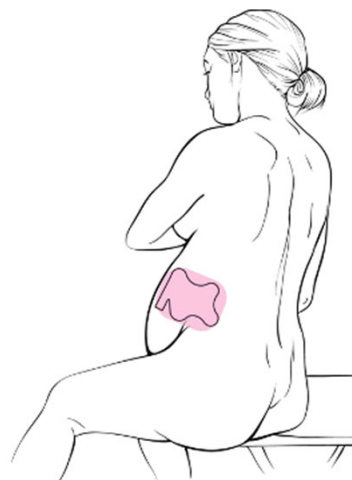
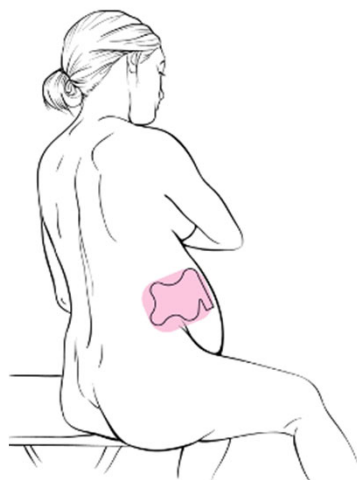


Place gel on gauze and rub into both side areas as shown above.

Wipe off excess gel with clean gauze over the entire abdomen. You are now ready to place patches.

MERIDIAN M110 INSTRUCTIONS FOR USE

PATCH PLACEMENT FOR MONITORING

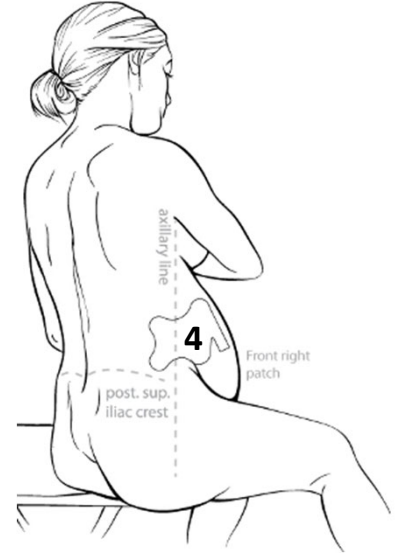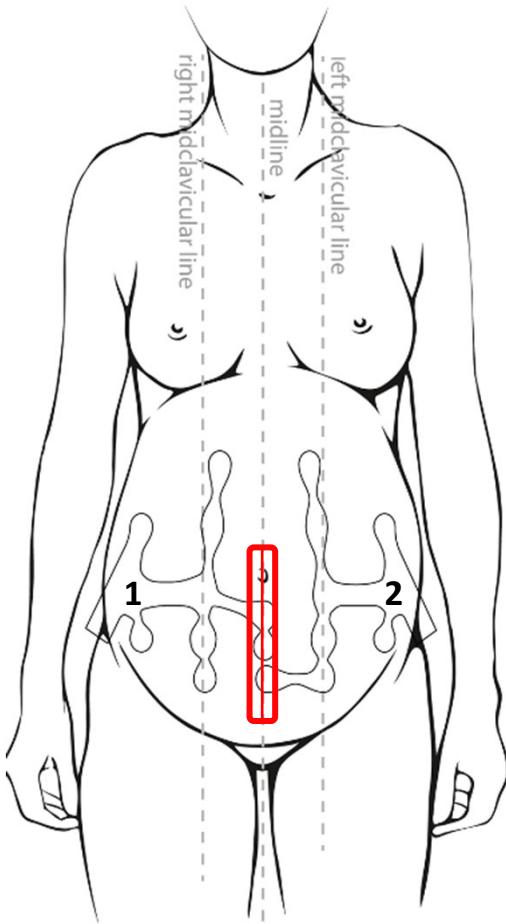

Place patches on skin using the order above.
Abdominal patches will form a line below the umbilicus as shown.


Place the side patch by aligning the silver tab on the side patch adjacent and parallel to the silver tab on the abdominal patch.
Leave 1-2 inches of space between the tabs.

CONNECTING PATIENT CABLES FOR MONITORING



Each cable has a pattern that corresponds with its matching patch. White patterns match with right patches and black patterns match with left patches.

When opening the latch on the cable connector, note how it does not open to 90 degrees.

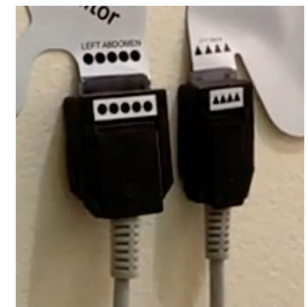

A small portion of the silver tab is visible even when it is fully inserted into the cable connector.