

Practices for the best monitoring experience

Exclusions for Existing Patient Conditions


- Multiple pregnancy
- Cardiac conditions: PVC and pacemakers
- Skin Conditions: open wounds or sores where electrodes may be placed, and silver allergy

Patient Preparation

- Ensure that any gel or lotion is removed from the abdomen. Prepare the skin with a warm wet cloth, assuring the skin is properly cleansed and degreased.
- Use Nu Prep Skin Prep lotion on abdomen, following Nu Prep's instructions for use

Troubleshooting (no FHR output)

- Minimize patient movement and/or rocking
- Verify all patch electrodes for good contact with the skin and all four connector's tabs are closed
- After 5 minutes of poor or no output, verify each electrode within the four patches for good contact with the skin then restart the device with a 5 second pause before powering back on

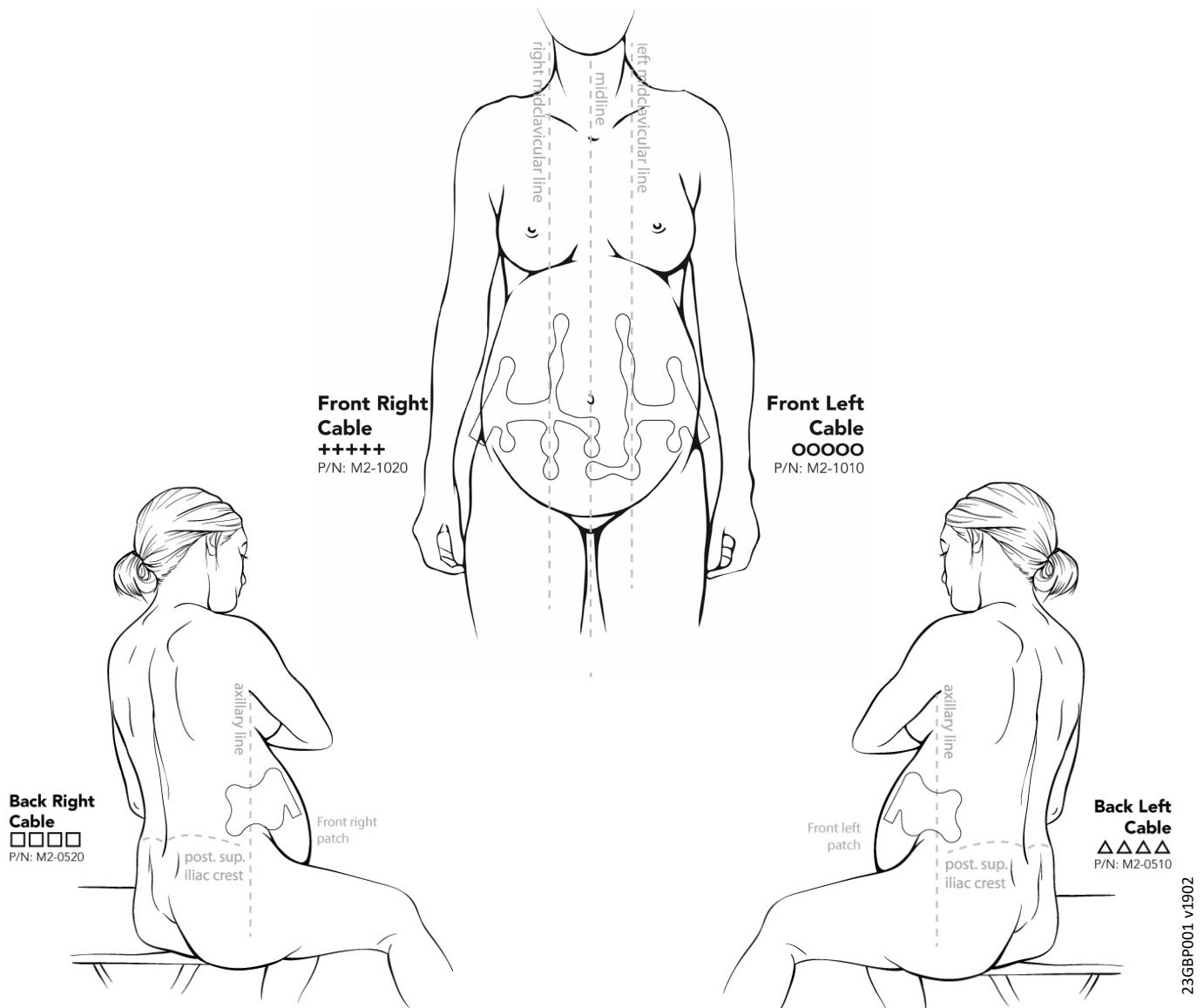

QUESTIONS - CONCERNS - ISSUES